

F&P Brevida™



NASAL PILLOWS MASK

Designed for Confidence. Built for Comfort.



Gaining patient confidence is key to successful CPAP therapy and patient confidence begins with a mask that fits and is comfortable.



Adjustable Headgear

Adjustable headgear which is designed to stay in place during sleep.

AirPillow™ Seal

The innovative AirPillow™ inflates to form a gentle and effective seal.

VisiBlue™

Blue highlights incorporated into the mask key components assist patients with daily mask use.

Washable Diffuser

The diffuser, which reduces noise and air draft, is washable and durable.



The innovative F&P Brevida™ AirPillow™ Seal

Two seal sizes

Extra Small-Small and Medium-Large.

Extra-thin, soft seal


Sealing region made of extra-thin (0.25 mm) soft silicone to provide a gentle seal.

Versatile pillows

Versatile pillow geometry to fit a wide range of nostril sizes.

Stabilizing side supports

Side supports designed to reduce the likelihood of the seal dislodging during sleep.

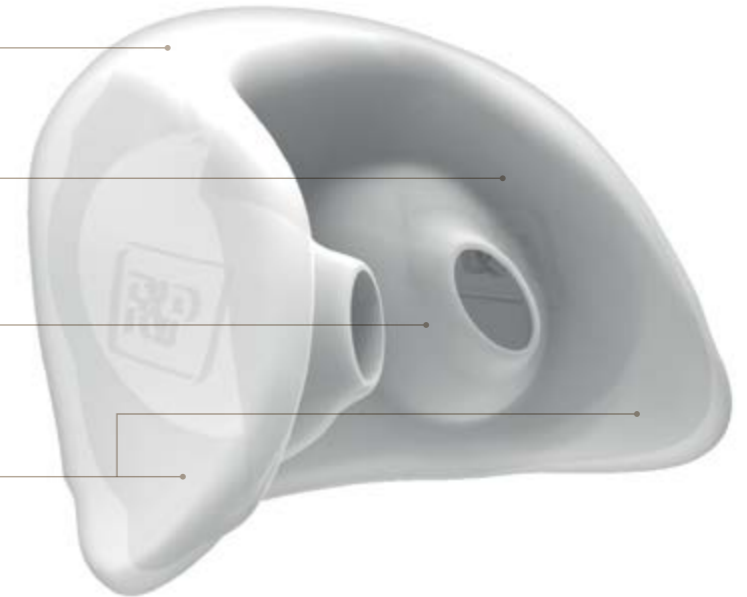


A gentle, soft seal

The AirPillow™ seal inflates to form a gentle 'pillow' of air and seals in and around the nose.


F&P AirPillow™ Nasal Pillows Seal

Seals in and around the nose

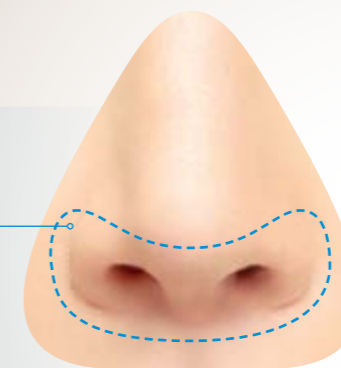

Traditional Nasal Pillows Seal

Seals directly in the nostrils

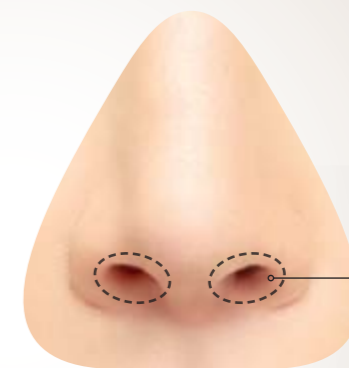

Fitting your F&P Brevida™



01

Step 1.
Hold the silicone seal away from your nose and use your fingers to carefully spread open the soft side supports of the silicone seal and then guide the two openings into your nostrils.



02

Step 2.
Hold the mask frame in position with one hand and pull the headgear over your head with the other hand, positioning the straps above the ears as per the fitting image above.



03

Step 3.
Gently tighten or loosen the headgear straps on the sides of the headgear until you achieve a comfortable seal.

NORTH AMERICA/ASIA-PACIFIC REORDER CODES

MASKS		400BRE114	Brevida Seal Medium-Large (50 pack)
BREISA	Brevida Extra Small-Small (one XS-S seal)	400BRE120	Brevida Headgear
BREIMA	Brevida Medium-Large (one M-L seal)	400BRE151	Brevida Headgear Clips (10 pack)
BREISMA	Brevida XS-S and M-L (two seals)	400BRE161	Brevida Diffuser 2 Pack
SPARE PARTS		400BRE171	Brevida Elbow
400BRE111	Brevida Seal Extra Small-Small	400FPH241	F&P Swivel 2
400BRE112	Brevida Seal Medium-Large	400BRE131	Brevida Mask No H/G XS-S
400BRE113	Brevida Seal Extra Small-Small (50 pack)	400BRE132	Brevida Mask No H/G M-L

EUROPE / AFRICA / SOUTH AMERICA REORDER CODES

MASKS		400BRE114	Brevida Seal Medium-Large (50 pack)
BREISU	Brevida Extra Small-Small (one XS-S seal)	400BRE120	Brevida Headgear
BREIMU	Brevida Medium-Large (one M-L seal)	400BRE151	Brevida Headgear Clips (10 pack)
BREISMU	Brevida XS-S and M-L (two seals)	400BRE161	Brevida Diffuser 2 Pack
SPARE PARTS		400BRE171	Brevida Elbow
400BRE111	Brevida Seal Extra Small-Small	400FPH241	F&P Swivel 2
400BRE112	Brevida Seal Medium-Large		
400BRE113	Brevida Seal Extra Small-Small (50 pack)		